


2. SELECTING CASES FOR INVISALIGN ASSIST

INVISALIGN ASSIST OFFERS SETUPS TO ACHIEVE YOUR TREATMENT GOALS

Complete anterior alignment and posterior corrections, required to support anterior alignment, can be achieved with Invisalign Assist. Before and after examples of setup outcomes - using Invisalign Assist - are provided below.

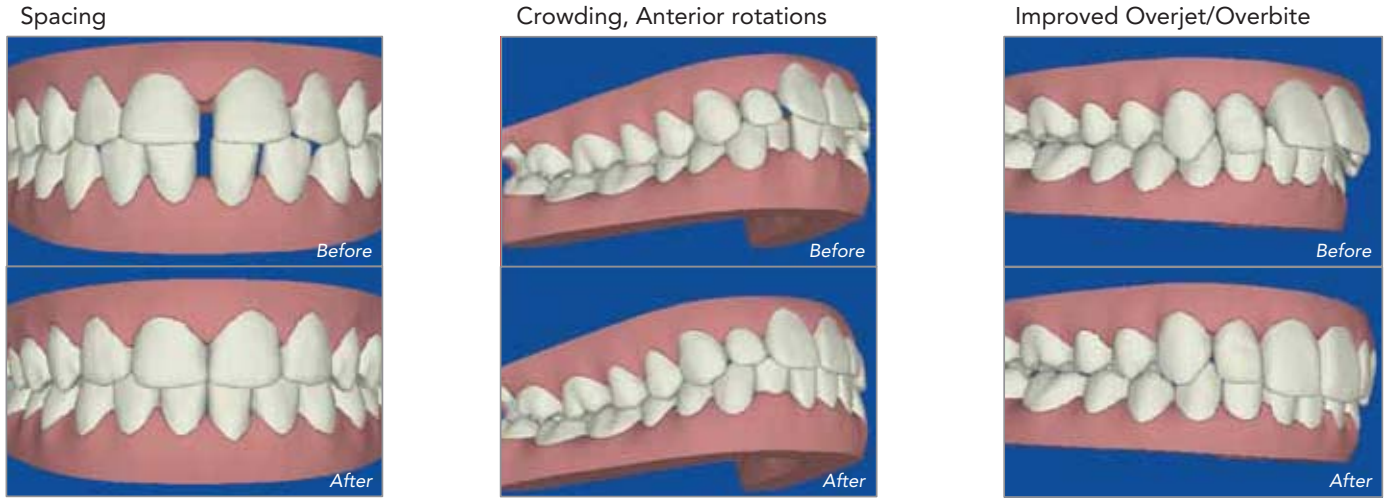


NOTE: A case may not qualify for Invisalign Assist when tooth movement exceeds product parameters, including:

1. Canine rotation greater than 45°
2. Anterior extrusion in excess of 2.5mm



INVISALIGN ASSIST CAN ALSO BE USED TO TREAT AND ADDRESS PATIENT'S CHIEF CONCERNS

Anterior alignment improvements can be made with Invisalign Assist setups, though a complete correction may not be shown. Before and after examples of compromised outcomes - using Invisalign Assist - are provided below.

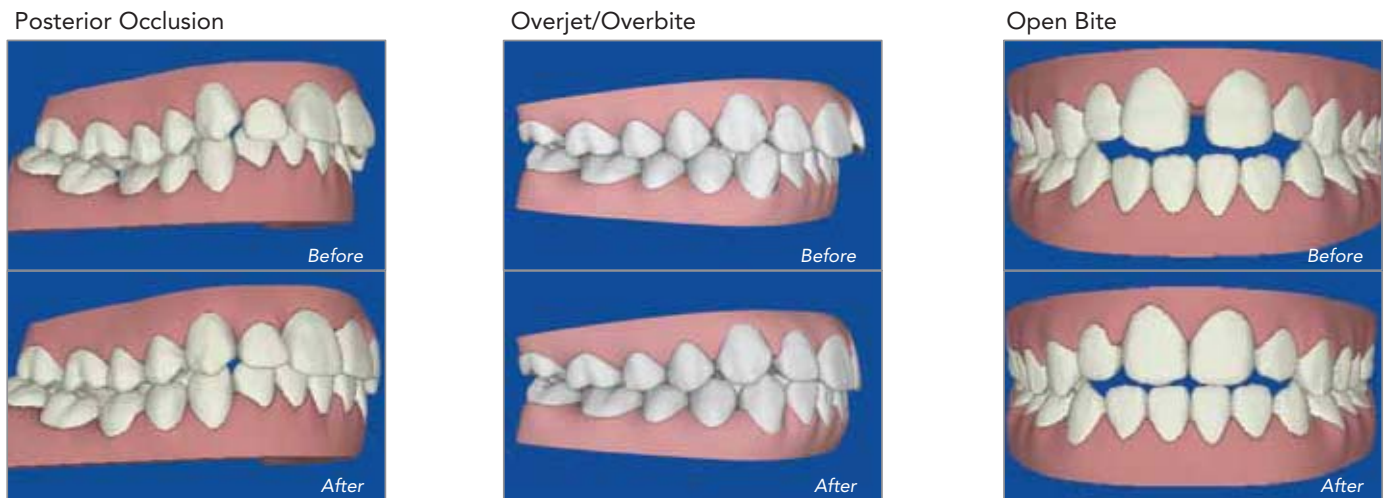

Actual patient outcomes may vary based on a number of factors such as physiology, patient compliance, and others.