

# 7. INVISALIGN ASSIST CLINCHECK® REVIEW

## TIPS & TRICKS

- ClinCheck software is required to preview all cases
- Install ClinCheck using the CD you received at training
- ClinCheck should be installed on all computers you plan to use/access for case submissions, treatment planning and patient consultation purposes
- You will be notified via email and an automated phone call when a ClinCheck is available for review
- The ClinCheck interface automatically adjusts for the product ordered

## LOG INTO VIP

- Go to [www.invisalign.com](http://www.invisalign.com)
- Select "I am a Doctor"
- Enter Username & Password

## PROCEDURE – CLINCHECK REVIEW FOR INVISALIGN ASSIST CASE

### 1 Review treatment plan

- IPR & Attachments
- Expected Number of Progress Impressions
- Review Aligner Features

### 2 Play treatment video from beginning to end

- View planned Rx and estimated duration
- Review the final position of setup
- Review Estimated Duration

### 3 Review model views

- Observe the five pre-set views of the model
- Use the video play buttons for each view

### 4 Review appointment plan task list

- View specific information for planned activities at each appointment

### 5 Open Comments pane

- Review Comments

### 6 Accept, modify, switch, or cancel ClinCheck

- Accept ClinCheck on Tx Plan tab, aligner fabrication will begin

