

2. SELECTING CASES FOR INVISALIGN® TREATMENT

TIPS & TRICKS

Most patients can be treated using the Invisalign family of treatment options. When clinicians initially begin treating patients with Invisalign, we suggest selecting cases that can be treated using the Invisalign Assist® option. Invisalign Assist is best suited for more aesthetically-oriented treatments without auxiliaries.

PROCEDURE – CASE SELECTION

- Using the illustrations below, answer the questions to determine the complexity of each case and recommendation on submitting.

Crowding




Is crowding, if present, combined with a narrow arch and/or lingually tipped teeth?

Spacing




Is spacing, if present, combined with a wide arch and/or flared anterior teeth?

Rotations

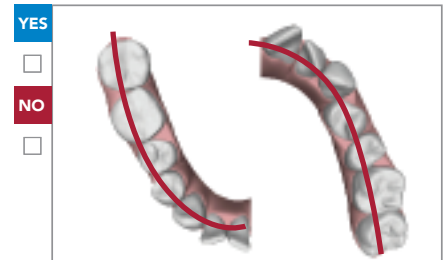


If rotations are required to align teeth, are rotations generally $\leq 40^\circ$?

Extrusions

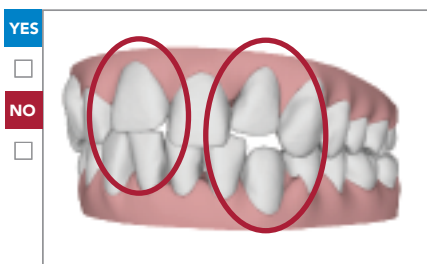


If extrusion of individual anterior teeth is required, is the needed correction 2.5mm or less?

Overjet

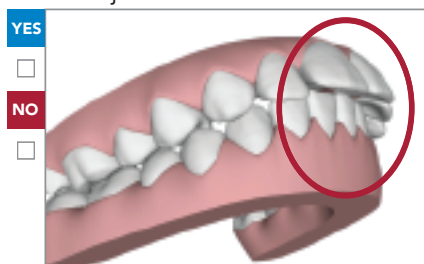

If overjet is excessive, is the needed correction 4mm or less?

Midlines

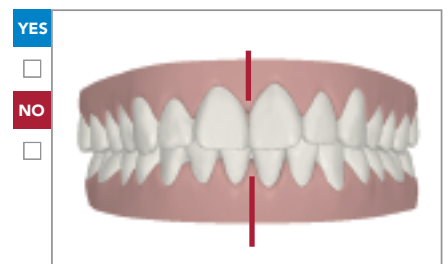

If midlines need to be shifted, is the needed correction 1mm or less per arch?

SUBMISSION RECOMMENDATION

- | | | |
|---------------------|----------------|-------------------|
| All YES | Simple Case | Invisalign Assist |
| Combo YES/NO | Moderate Case | Invisalign Assist |
| Mostly NO | Difficult Case | Invisalign Full |

If a case does not qualify for Invisalign Assist, you may switch to Invisalign Full, however additional fees may apply. More advanced skills may be required for Invisalign Full treatment.