

# USING INVISALIGN® DETAIL PLIERS

## PROCEDURE

1. For mesial in and distal out rotation.
2. Create a dimple Mesial-buccal.
3. Create a dimple Distal-lingual.
4. Dimples should look like this.
5. Fill each dimple with Clear Loc™ adhesive.
6. Light-cure 20 seconds.
7. Check interproximal contacts.  
If they are tight carefully relieve them.
8. Insert the aligner.

Note: steps #5 and 6 are optional tips in order to provide extra strength.

