

SUPPORT YOU WANT. Patient outcomes you expect.



INVISALIGN ASSIST

BY OFFERING PRODUCT SUPPORT THROUGHOUT THE TREATMENT PROCESS, INVISALIGN ASSIST® HELPS YOU ACHIEVE THE PATIENT RESULTS YOU WANT FOR ANTERIOR ALIGNMENT AND AESTHETICALLY-ORIENTED CASES.

PROGRESS TRACKING

Helps Keep Teeth On Track

APPOINTMENT PLANS

Guidance For Every Appointment

IMPROVED CLINICAL PROTOCOLS

ClinCheck Treatment Plans Developed For You

COMPLIANCE INDICATORS

Helps Determine If Patients Are Wearing Aligners Regularly



Learn more at www.invisalignassist.com

PROGRESS TRACKING

Helps Keep Teeth On Track

Progress Tracking identifies teeth not moving to plan and adjusts accordingly.

With Invisalign Assist, aligners are shipped in batches. When you distribute your patient's 10th aligner, a Progress Tracking impression should be taken and submitted to Align. The impression is compared to the original treatment plan and identifies teeth not moving to plan. Any corrections are built into your next batch of aligners.

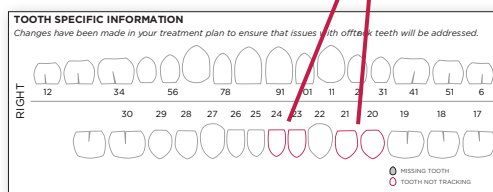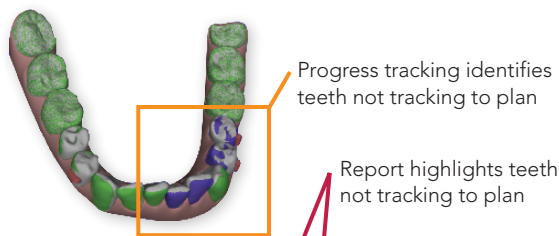


APPOINTMENT PLANS

Guidance For Every Appointment

Appointment plans provide task-level guidance for every patient at each appointment.

Invisalign Assist appointment plans are customized for each patient and include specific and detailed tasks to be performed at each appointment.

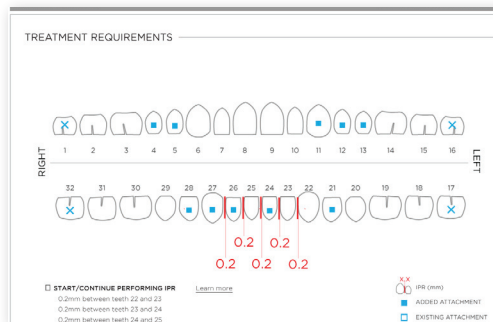


IMPROVED CLINICAL PROTOCOLS

ClinCheck® Treatment Plans Developed For You

ClinCheck treatment plans display detailed tooth movements designed to reach your goal. Included in each treatment plan, when needed, are NEW SmartForce™ features—engineered to deliver more predictable tooth movements.

Adjustments can be made to final position

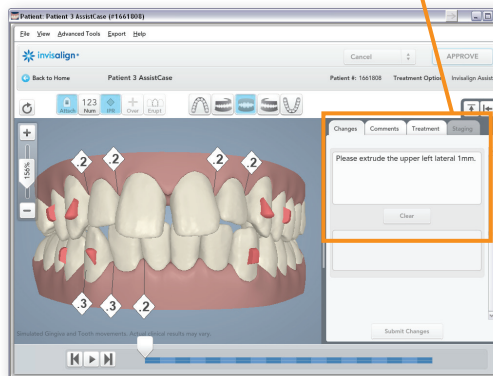

COMPLIANCE INDICATORS

Helps Determine If Patients Are Wearing Aligners Regularly

Compliance indicators encourage the patient to be more diligent about compliance. Wear indicators help doctors gauge if patients are wearing aligners long enough to achieve the desired outcome.

